

# lifeinabox

*Live your life*



Instruction manual



## WELCOME TO THE LIFEINA FAMILY

---

LifeinaBox is a product of love and happiness. We have really tried to make a device as perfect as possible. And we love to continuously improve our products, so if you have any comments or suggestions, please do not hesitate to contact us on [hello@lifeina.com](mailto:hello@lifeina.com).

Please read these instructions carefully before using the device.

## WHAT IS LIFEINABOX?

---

LifeinaBox is a miniature fridge for the safe transport and storage of fragile medication. LifeinaBox has an internal temperature controller that turns the unit on when the internal temperature reaches 8°C (46°F) and turns it off when it falls to 2°C (36°F). LifeinaBox will not prevent medicine from freezing if the ambient temperature falls below 0°C (32°F).

## USING LIFEINABOX

---

LifeinaBox is designed for simple, easy operation. Simply plug the correct power connector (car, mains or battery) into the back of the device, wait for the temperature to settle (about 15 minutes), and place your medication inside. All other functions are controlled with the Lifeina App.

LifeinaBox is shipped with a 12V DC connector (for the car) and a 110/240V AC adapter (for the house). Separate Lifeina battery packs are also available as independent purchases. Modifying the power cable or using a cable that was not supplied with LifeinaBox can damage the unit, jeopardize your medicine's effectiveness and void your guarantee.

If you are using LifeinaBox for glass medicine vials, we recommend that you place padding material between your medicine vials to protect them against accidental breakage.

## **CONNECTING TO THE LIFEINA APP**

Download the Lifeina app on your smartphone (iOS or Android) and set up a user account.

- 1) Switch on the Bluetooth on your smartphone, and make sure it is connected to the internet.
- 2) Connect LifeinaBox to a power source.
- 3) Open the Lifeina app on your smartphone and sign in. Place LifeinaBox near your smartphone and follow the instructions on the app.

## **CARE AND MAINTENANCE**

- Do not drop, modify or disassemble the device.
- Do not bend or twist the power cables.
- There are air vents on the sides and under the device. These vents must be kept free for air to circulate. Do not connect LifeinaBox to a power source when it is in a closed area such as packed luggage or a desk drawer.
- LifeinaBox should never be submerged in liquid or exposed to excessive moisture. Use a cloth moistened with water or neutral detergent to clean the device.
- Avoid high temperatures, direct sunlight, high moisture or dust.

## **THE LIFEINA QUALITY GUARANTEE**

We guarantee LifeinaBox for one year from date of purchase by the customer. During this period, Lifeina assures free repair of internal faults or manufacturing errors. If you experience a problem with your LifeinaBox, please pack it carefully and send it to your local distributor, together with the guarantee card at the end of this booklet.

Unfortunately, our guarantee does not cover damage caused by dropping or impacts, by faulty handling or the intervention of a third party, by natural disasters, or by water.

## TECHNICAL SPECIFICATIONS

---

Dimensions:	20 (W) x 10 (D) x 9 (H) cm
Capacity:	367 cm <sup>3</sup>
Net Weight:	770 gm
Voltage:	12V DC (car) or 110-240V AC
Power consumption:	25 watts max
Temperature range:	2°C to 8°C, +/- 1°C
Insulation:	CFC-free polyurethane foam
System:	Wear-free thermoelectric technology with interior semiconductor fan
Material:	ABS

*Specifications are subject to technical improvement, changes or modifications without notice.*



[www.lifeina.com](http://www.lifeina.com)

**Lifeina**  
6 rue de Castellane  
75008, Paris, France  
[hello@lifeina.com](mailto:hello@lifeina.com)