QUICK USER GUIDE

CHARGING THE BATTERY

The LifeinaBox battery needs to be fully charged before you start using the device

- 1) Make sure the device is connected to the power ①
- 2) Flick the O/I switch **under the device** to the ON position ②
- 3) Press the button **on the side** of the device twice to switch it off ③
- 4) Leave it to charge for two hours

USING LIFEINABOX

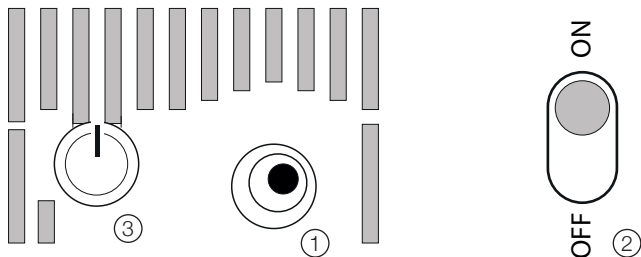
- Connect it to the power and fully charge the battery (see above)
- Switch on LifeinaBox by either:
 - 1) Pressing the button on side of device once ③
 - 2) Unplugging the device and plugging it back in
- Wait for LifeinaBox to reach a stable temperature (about 15 minutes)
- Place your medication inside
- To use the Lifeina application, simply download the app and turn it on within distance of your LifeinaBox

NOTES

- If you are planning on using LifeinaBox at home, switch off the battery (under the device ②) so that it lasts longer. But don't forget to fully charge it before a trip
- If you want to use the battery mode, flick the O/I switch under the device to the "ON" position
- Be happy

Please do not block the air vents on the side and under the device

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